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---

# Content

	Page
<b>1. Important information</b>	<b>10</b>
1.1 Liability and Warranty	10
1.2. User groups	10
<b>2. Information on the safety and protection of the product</b>	<b>10</b>
<b>3. Product description</b>	<b>11</b>
3.1 Application scope	11
3.2 Technical Data	11
3.3 Optional accessories	12
<b>4. Assembly</b>	<b>12</b>
4.1 Assembling the safety bars	12
4.2 Assembling the straps	13
4.3 Anchor points	13
4.3 Disassembly	13
<b>5. Application</b>	<b>14</b>
5.1 Safety information	14
5.2 Placing the injured person on the ultraROLL-STRETCHER	15
<b>6. Care and maintenance</b>	<b>15</b>
<b>7. Storage</b>	<b>15</b>
<b>8. Maintenance</b>	<b>15</b>
<b>9. Service life</b>	<b>16</b>

---

## 1. Important information

Before using the stretcher, please carefully read the operating manual and safety instructions. Keep the operating manual in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealership or ultraMEDIC GmbH & Co. KG directly. Always specify the serial number to identify the stretcher.

### 1.1 Liability and Warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts. Shipping or wage costs are not subject to warranty.

Liability for damages caused by transport, operating errors or non-compliance to the operating manual is excluded.

The serial number is specified on the sticker attached to the side of the ultraROLL-STRETCHER. In the event of a warranty claim, always specify this number, the date of purchase and the name of the dealership.

### 1.2 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the ultraROLL-STRETCHER.

## 2. Information on the safety and protection of the product

- Before and after every use, check the components and transport straps for damage and completeness. A detailed list of parts to be inspected is contained under item 7.
- Ensure that the patient restraint straps are installed correctly and that the anchor points of the transportation gear are firmly seated.
- Before initial use, conduct a test run with an empty stretcher.
- In the event of damage or suspected damage, remove the product from operation immediately until any occurring irregularities have been remedied.

---

## 3. Product description

### 3.1 Application scope

The ultraROLL-STRETCHER is a rollable stretcher used to transport injured parties from difficult-to-access or narrow areas. It can be used for any kind of rescue mission, in high or low places, as well as for all kinds of horizontal and vertical rescues.

The ultraROLL-STRETCHER is not a spine immobilisation device.

The user is responsible for transport certification pertaining to the respective helicopter model.

Where there is reason to suspect spinal injury, the injured party must first be placed on a spinal board or on a vacuum mattress and immobilised.

### 3.2 Technical Data

Order number:	SAN-9000
Designation:	ultraROLL-STRETCHER
Weight:	7.3 kg (without carrier bag)
Dimensions:	200 x 60 x 30 cm
max. load capacity:	300 kg
Operation temperature:	-30°C to +50°C

Scope of delivery:	8 padded handles (SAN-0260-GR)
	4 patient restraining belts with COBRA latches (SAN-9000-G-1)
	1 retractor belt (SAN-900-G-3)
	2 adjustable foot element fixation straps (SAN-9000-G-2A / -2B)
	1 water-tight rucksack carrier bag (SAN-9000-T)
	1 base plate
	5 safety bars:
	1 footrest with 2 straps, 270 x 200 mm,
	<i>3 horizontal bars:</i>
	1 leg protector, reinforcement 390 mm, 690 x 100 mm;
	1 pelvis protector, reinforcement 460 mm, 690 x 100 mm;
	1 thorax protector, reinforcement 510 mm, 690 x 100 mm;
	<i>1 longitudinal bar:</i>
	1 spine protector, reinforcement 910 mm, 135 x 1050 mm.

### 3.3 Optional accessories

Order no.	Designation	Image
SAN-0086	ultraBRIDLES Length-adjustable transportation gear	
SAN-9005-H	ultraFIX-HORIZONTAL Horizontal transportation gear	
SAN-0097	ultraFLEX Flexibly adjustable transportation gear	
SAN-9005-V	ultraFIX-VERTICAL Fixed transportation gear for vertical suspension	

Additional supplementary products can be viewed on our Internet page under [www.ulframedic.de](http://www.ulframedic.de) or in our catalogue. .

## 4. Assembly


The ultraROLL-STRETCHER is not delivered in pre-assembled state.

Observe the following steps during assembly:

### 4.1. Assembling the safety bars:

1. Remove the contents from the carrier bag. Ensure that all parts are available in accordance with the delivery scope.
2. Bend back the base plate so that it can lie flat on the ground.
3. Assemble the foot support and horizontal bars. When assembling the horizontal bars, observe the difference in reinforcement length. The shortest reinforcement is assembled first following the assembled foot support, the longest reinforcement of the horizontal bar is located underneath the head support.

- 
4. Push the longitudinal bar under the horizontal bar to the thorax protector and insert through the 2 straps up to the stop. The lower part of the longitudinal bar remains in place above the horizontal bar to the pelvis protector. (refer to Image).

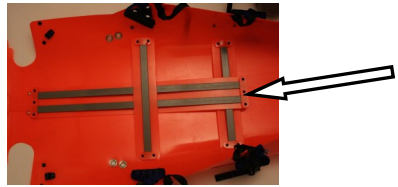#### 4.2. Assembling the straps

##### Patient restraint straps

Patient restraint straps are attached to the individual, metal-enforced eyelets at both sides of the ultraROLL-STRETCHER.

1. Open the buckle and position the strap sections on both sides of the ultraROLL-STRETCHER.
2. Thread the rotated eye of the patient restraint strap into the eyelet.
3. Pull the strap through the rotated eye completely.

##### Foot section restraint straps

Ensure that the buckle of the foot section's restraint straps is positioned on the outer side of the ultraROLL-STRETCHER.

1. Take up the strap section with the buckle. Thread the rotated eye through the metal-enforced eyelet and the opening of the plastic eyelet.
2. Pull the strap through the rotated eye completely.
3. Secure the other half of the strap into the closest metal-enforced eyelet on the base plate.
4. Guide the end of the strap through the buckle and pull the strap tighter.

##### Handles


Handles are attached to the 6 anchor points in the metal-enforced eyelets.

The same eyelets are intended for the handles and for the transportation gear. Both carrying aids can be secured and used in one eyelet at the same time.

---

### 4.3 Anchor points

Anchor points for horizontal and vertical suspending are located at both sides of the ultraROLL-STRETCHER.

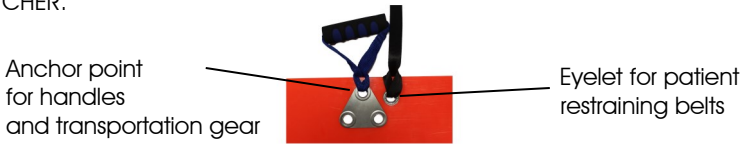### 4.4 Disassembly

1. After use, only remove the spine support, all other parts (horizontal bars, foot support, straps and handles) can remain assembled.
2. If the ultraROLL-STRETCHER is soiled after use, read item 6 in the operating manual.
3. Place the ultraROLL-STRETCHER on a flat surface. Start with rolling up the head end.
4. Tighten the belt retractor around the rolled up stretcher.
5. Place all parts into the carrier bag.

## 5. Application

### 5.1. Safety information

- A specialist medical assessment must be conducted before the injured party is moved, lifted or transported.
- The ultraROLL-STRETCHER is not a spine immobilisation device.
- Where there is reason to suspect spinal injury, the injured person must first be placed on a spinal board or on a vacuum mattress and immobilised.
- At least 2 users are required to operate the ultraROLL-STRETCHER to ensure the patient's safety.
- The injured person must be immobilised on the ultraROLL-STRETCHER during every use. Insufficient or no restraint may lead to severe patient or rescuer injuries.
- Never leave the injured person on the stretcher without constant supervision.

---

## 5.2. Placing the injured person on the ultraROLL-STRETCHER:

1. Spread out the ultraROLL-STRETCHER as flatly and as close to the patient as possible.
2. Position the ultraROLL-STRETCHER underneath the patient. Ensure that the feet are positioned above the footrest.
- 2a. In the event of spinal injury, first place the injured person onto a spinal board or a vacuum mattress. Leave the immobilisation device underneath the patient. Use straps to secure this immobilisation device to the ultraROLL-STRETCHER into a securely fixed unit.
3. Use the foot straps to bring the foot plate into vertical position.
4. Secure all 4 patient safety straps. Ensure that the injured party's breathing is not encumbered by the traction of the straps.
5. Close the ratchet buckle near the head up to the stop.
6. Check all buckles and straps before lifting and transporting the patient.

## 6. Care and maintenance

### Moisture:

Moisture can be removed with a dry, soft cloth.

You can also hang up the ultraROLL-STRETCHER to dry.

The ultraROLL-STRETCHER may not be subjected to direct sunlight or heat radiation.

### Soiling:

The soiled ultraROLL-STRETCHER can be cleaned with a moist cloth or sponge. You can use a brush and clean water or mild soapy water to clean the straps.

### Disinfection:

For disinfection, you can use commercially available, alcohol-based agents. Do not use any other chemical cleaning agents or solvents.

## 7. Storage

Store the ultraROLL-STRETCHER in a cool, dry and dark place in its designated carrier bag. Avoid direct sunlight and direct heat radiation from heat sources. Continuous heat and direct sunlight can damage the material.

During storage, do not place heavy material on top of the ultraROLL-STRETCHER.

## 8. Maintenance

The stretcher will age in accordance with the frequency and intensity of its use. In this regard, correct use is assumed.

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraROLL-STRETCHER, the transportation gear and straps visually inspected by an expert on an annual basis.

For German users, we recommend consideration to the following regulations and guide-

---

lines:

MPBetreibV  
DGUV Principle 305-002 (previously GUV-G 9102)  
DGUV Regulation 112-198/199 (previously BGR/GUV-R 198/199 ).

The following parts must be inspected:

Base plate and base plate straps	Carbines	Straps and handles	Rivet connections
Deformation	Function	Scouring points, cuts	firm seat
Cracks and tears	Deformation and torsion	Seams	
	Cracks and tears	tissue-damaging stains	
		Other damages	

In the event of damage or suspected damage, remove the product from operation immediately until any occurring irregularities have been remedied.

## 9. Service life

The service life of the ultraROLL-STRETCHER depends on the individual conditions of use. The stretcher will age in accordance with the frequency and intensity of its use. In this regard, correct use is assumed.

The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition or **no later than after 12 years**.

The lifecycle of the optional accessories is described in the associated operating instructions.

**Impressum:**

ultraMEDIC GmbH & Co.KG

Im Bruch 11-15

D-56567 Neuwied

Tel. (02631) 96983-0

Fax (02631) 96983-33

[info@ultramedic.de](mailto:info@ultramedic.de)

[www.ultramedic.de](http://www.ultramedic.de)